



正面

### LET'S GET STARTED

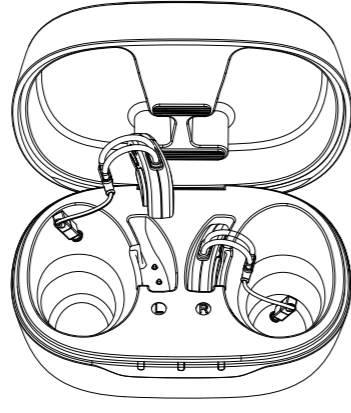


#### 1 ACTIVATE YOUR DEVICE

1. Remove the stickers.

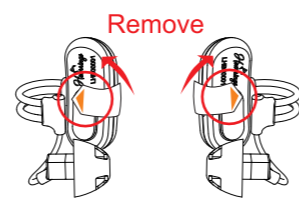


2. Charge the device to activate. (For the best results, fully charge before first use.)

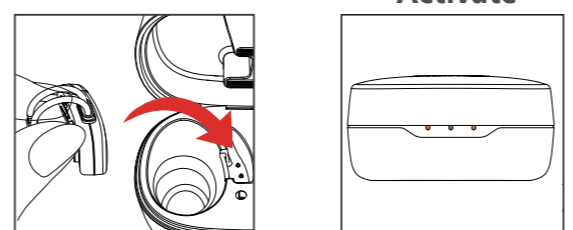


#### 2 WEAR YOUR DEVICE

1. Identifying left and right.

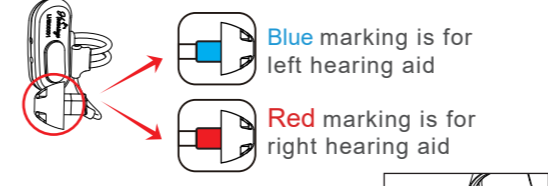

2. Device turns on automatically when removed from the charging case.

3. Secure the hearing aid behind your ear, then gently insert the eartips into your ear canal while holding the ear tube.

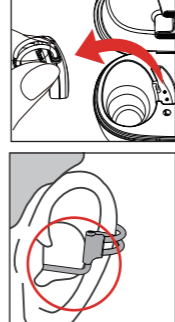


4. Place in charging case to power off automatically.


#### 3 HOW TO CHANGE THE EARTIPS

Incorrect eartip sizing may cause whistling and feedback. Try a different size to reduce feedback.


1. Remove the eartips.

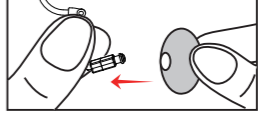

2. Choose the proper size.

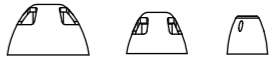


3. Attach preferred tip.

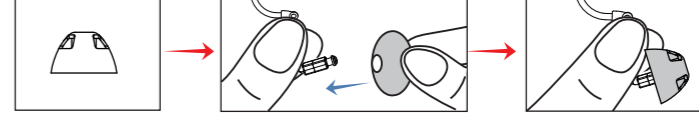

**Note:** Please pull the eartips gently to check if they are attached securely.

背面

#### 4 HOW TO ADJUST YOUR DEVICES

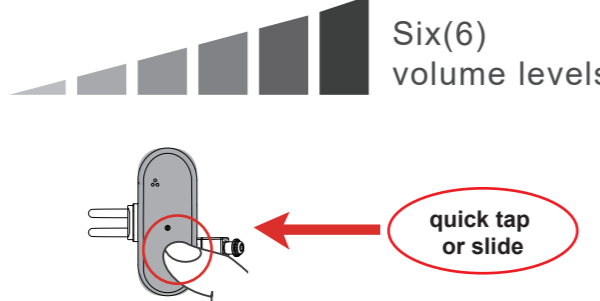


Six(6) volume levels

Action	Prompt tone
Volume Up	Beep
Volume Max	Beep Beep Beep

Press the button to cycle volume levels.


#### 5 PROGRAM SWITCH

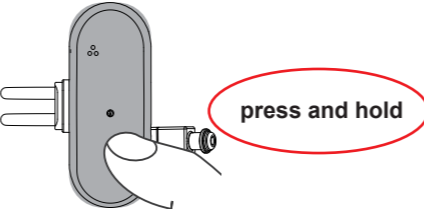

Hearing Program	Prompt Tone
1 Normal	program 1
2 Restaurant	program 2
3 Outdoors	program 3
4 Music	program 4

#### 6 HOW TO CHARGE


1. Insert the hearing aids into the case to charge.

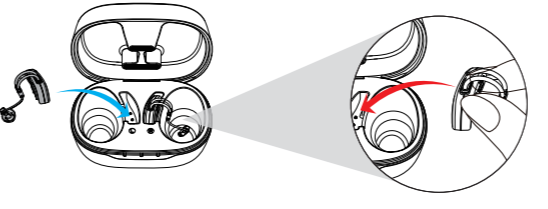

Charging status (Both sides)	
	The orange lights indicate charging.
	The green lights indicate a full charge.

Charging status (Middle)	
	The orange lights indicates charging.
	The green lights indicates a full charge.

2. Plug in charging cable to charge the charging case.

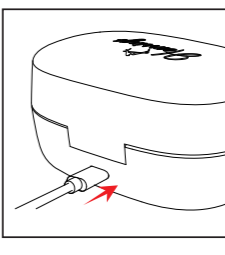

折痕，不印刷出来

折法

